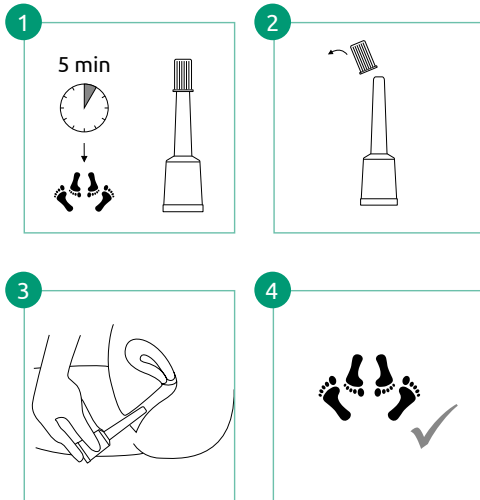
# Prefert®

Vaginal Gel for couples trying to conceive

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## Prefert® Vaginal Gel for couples trying to conceive

### Mode of action:

Prefert® Vaginal Gel creates an environment in which sperm has the best chances of reaching a woman's eggs.

Its osmotic properties provide optimum room for manoeuvre. This boosts sperm motility and is the ideal jumpstart for sperms to successfully find their way to a woman's eggs.

With a neutral pH of 7.2, Prefert protects sperm from too acidic an environment, thereby ensuring their survival.

In comparison to conventional, water-soluble lubricants, Prefert® Vaginal Gel improves sperm motility.

### When to use:

For temporary use on the three most fertile days of the female cycle (time of ovulation).

Use only once within 24 hours.

### Ingredients:

Purified Water, Sodium Chloride, Sodium Phosphate, Potassium Phosphate, Arabinogalactan, Hydroxyethylcellulose, Polyhexanide

### Application:

Introduce 5 minutes before sexual intercourse.

1. Prior to intercourse, remove a disposable Prefert® Vaginal Gel applicator from the pack
2. Remove the cap.
3. Insert the vaginal applicator into the vagina. Press the applicator together to introduce the gel. Remove the applicator.
4. You may now have sexual intercourse.

The contents of the pack are sufficient for 8 applications.

Discard the empty applicator in household waste after use.

Free of parabens.

### Information:

If Prefert® Vaginal Gel causes you any irritation when you use it, stop using it and ask your doctor.

Keep out of the sight and reach of children.

Do not use Prefert® Vaginal Gel if you have recently had vaginal surgery or if you have an acute vaginal infection.

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KESSEL medintim GmbH